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# Instructions for use

## INSERTION

- ① "R" mark is for the right ear.  
Position it as shown

- ② Insert the earplug into the ear canal

- ③ Press the top of the earplug into the concha

## REMOVAL

- ① Position your finger at the bottom of the ear canal as shown

- ② Gently push your finger so that the earplug comes out of your ear

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**EN** User instructions  
Earplugs for water

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For more information,  
please contact:

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If your earplugs have a cord, **NEVER remove them by pulling on the cord** as this can damage the product.

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## EN - User guide - Earplugs for water

The hearing protection is custom-made according to the ear impression. Please read the instructions for use carefully before starting to use your hearing protection.

### Insertion and removal of earplugs

The earplug marked with "R" corresponds to your right ear. Insertion: Grasp your ear protection with your thumb and index finger (using the hand on the same side as your ear) so that the "R" marking is facing forward. While keeping the marking in this position, insert the earplug into the ear canal. Then push the earplug into your ear with your index finger. During this step, you can open your mouth to make insertion easier. Ensure that all parts of the earplug are correctly placed in the folds of the outer part of the ear. Removal: Position your index finger at the bottom of the ear canal between your tragus and antitragus. Gently push with your index finger to remove the earplug from the ear. Then grasp your ear protection with your thumb and index finger. **For earplugs with a connecting cord: Warning: Do not pull on the connecting cord to remove the earplugs.**

### Water tightness

Always place the hearing protection so that the ear canal is properly blocked. Position the hearing protection correctly so that the water tightness is guaranteed.

### Storage and Maintenance

The hearing protection is reusable. It should be cleaned after each use. Cleaning with soap and water. Chemical detergents and compressed air should be avoided as they are harmful to the product. Never heat the hearing protection to dry it. Let it dry at room temperature. The life of the hearing protection depends on the use and maintenance. We recommend storing the hearing protection in the case included in the delivery for storage and transport.

### Skin irritations and dysfunctions

The hearing protection is made of hypoallergenic and biocompatible medical silicone material. It is coated with a hypoallergenic lacquer. Consult an ENT specialist in case you still have itching or allergies. This may be an over-reaction of the skin against the material used. Please contact your dealer or the manufacturer in the event of incorrect positioning of the protection or if the sealing effect of your hearing protection is insufficient. If the hearing protection is inserted incorrectly or if the instructions for use are not followed, the effectiveness of the hearing protection may be impaired.

### Warnings

Hearing protection is not made for scuba diving. It can prevent pressure balance. The hearing protection is for personal use only and should not be worn by another person. The retailer and the manufacturer accept no liability for a possible reduction in water tightness and the associated consequences. Abrupt movements of the head may cause a possible reduction in water tightness. Hearing protection cannot be used as part of noise protection. Water hearing protection is a comfort hearing protection and is not subject to personal protective equipment certification. It is the responsibility of the user to check the suitability for the intended use, taking into account any influences that may affect performance. Warranty and liability claims for our products are provided for in the general terms and conditions of sale, unless otherwise stipulated by law.